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Instructions for Use

1. Press the mask against the face with the metallic strip uppermost
2. Place the bands around both ears
3. Mould the metallic strip over nose bridge and the mask should snugly over the face
4. Press the mask against the face with both hands to seal air leaks

PRODUCT FEATURES

- Efficient filtration: Filtering rate above 94% with pollen, droplet, dust, and smog effectively
- 5 Layers purification: Filament non-woven fabric, flexible moderate thickness, adsorption particle, double layer melt-blown filters
- Smooth breathing: Flexible insert nosebar ensures optimal sealing between the bridge of the nose and the mask
- Ergonomic design: Comfortable to wear, low impedance filter material and no odor
- Secure seal: Soft metal nosebar for best seal between nasal beam and mask

WARNINGS

1. Check the expiry date and mask before first use
2. Do not use after expiry date or the mask damaged
3. Do not use a microwave oven to heat it
4. Check and ensure the mask and straps were fit properly and tightly
5. The mask can not use in the space with poor ventilation due to the oxygen deficiency (below 19.5% vol)
6. Stop use mask when breathing is very difficult and change a new one
7. In order to keep the mask clean, avoid touching the inside of the mask with your hands
8. Not recommended for children under three years old due to low lung capacity
9. Please store the respirator in a temperature range of -20°C to 25°C and a relative humidity of less than 80%.

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